OrthoP PreventAligner®

Kll-**W** soft

D

GB

F

I

E

Patienteninformation

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D
GB
F
I
E

Patient information

Prevention-oriented early treatment

Harmful factors that have a lasting effect on facial growth and tooth positions require **early preventive treatment, i.e. prophylactic treatment.**

Impairments include harmful sucking habits, dysfunctions of the oral musculature - often associated with general postural weaknesses.

With the **OrthoPreventAligner®**-system, dysfunctions such as open mouth posture, incorrect tongue rest positions, faulty swallowing and a muscularly weak lip closure are corrected. These are often associated with the lower jaw distal bite (KL II). This also includes gapped and protruding incisors as well as frontally open bites, which result in speech disorders, restricted biting or chewing. When the mixed dentition is treated early, later therapy is usually unnecessary or considerably reduced in severity.

How jaw anomalies develop

There are congenital, inherited and acquired anomalies in the maxillofacial region.

Jaw anomalies, as well as the causes and effects of the predominantly acquired malocclusions can be effectively treated with prefabricated appliances such as the **OrthoPreventAligner®** system.

These anomalies are usually caused by non-physiological influences on the developing deciduous dentition, by sucking habits and muscular dysfunctions. Causative malfunctions and jaw anomalies that have already developed are usually transferred to the permanent dentition and, if left untreated, become even worse.

The tongue plays an essential role in the development and maintenance of dental anomalies: It is displaced from its resting position in the upper jaw, or becomes lodged between the rows of teeth. This is how jaw anomalies develop, which often remain for life.

The OrthoPreventAligner® KII-W

The **OrthoPreventAligner® KII-W** (KL II = lower jaw distal bite) is made of a highly elastic silicone. The lip shield for the upper jaw is extended and strongly tilted backwards. This completely encloses the incisors, which are usually tilted forward, and the alveolar extension.

The effect of the **OrthoPreventAligner® KII-W** to eliminate the incisor step is partly **indirect**: The stimulation of growth of the incorrectly positioned lower jaw leads to its regular position to the upper jaw and to the elimination of the incisor step. In addition, there is a **direct** effect by resting the upper lip shield on the incisors, which usually protrude with gaps. This directs them towards the palate and thus contributes to the reduction of the incisor step.

Usually, the removal of the incisor step not only improves pronunciation, but also makes it easier to bite with the incisors.

KII-W Mixed dentition

In the case of an open bite with prolonged vestibular pelotte
(Similar to Fig.)

Development of lower jaw distal bite and an incisor step

If the incisors do not bite over each other like scissors and if, more or less, there is a large step between the teeth of the upper jaw and those of the lower jaw, the upper teeth **appear** to be too far forward.

In fact, however, the entire lower jaw is too far back.

Every person is born with a "baby's distal bite". For developmental reasons, most people compensate for this by themselves in the first year of life - but in a quarter of newborns, they either don't completely compensate or not at all.

Irrespective of this, the upper incisors are often tilted further forward due to the insertion of the lower lip and are additionally strengthened there by the **incisor step**.

Since there is still general growth in children, the growth deficit can be compensated for by regularly wearing functional orthodontic appliances (including the **OrthoPreventAligner® KII-W**).

Recommended wear

In order to break harmful habits and to eliminate **mal-functions** as well as to classify "crooked" teeth, the **OrthoPreventAligner®** should be worn regularly for at least 2 hours a day and at night.

Wearing check

	Week 1		Week 2		Week 3	
	 hr.	 hr.	 hr.	 hr.	 hr.	 hr.
MON						
TUE						
WED						
THUR						
FRI						
SAT						
SUN						

Instructions for use

Rinse the **OrthoPreventAligner**[®] with warm water before using for the first time. The **OrthoPreventAligner**[®] can be disinfected briefly by boiling water after infections.

Cleaning instructions

Clean thoroughly with a toothbrush under warm running water. Alternatively, OROFAN[®] dental cleaning tablets can be used.

The **OrthoPreventAligner**[®] is not dishwasher or microwave safe. Only clean the hygienic **OrthoPrevent Aligner**[®] box with a damp cloth.

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