OrthoP PreventAligner®

DB-W

D

GB

F

I

E

Patienteninformation

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D
GB
F
I
E

Patient information

Prevention-oriented early treatment

Harmful factors that can have a lasting effect on facial growth, tooth positions or the periodontium require early preventive, i.e. **prophylactic, treatment**. Beyond harmful sucking habits and dysfunctions of the oral musculature, preventive intervention may also be necessary for extremely deep bites caused by certain cranial structures.

With the **OrthoPreventAligner®** system can treat **dysfunctions** such as an incorrect resting position of the tongue. It is often also associated with a backward position of the lower jaw and very deep overbite with **contact** or a **bite into the palatal mucosa** of the upper jaw.

Such often painful bites, into the gums of the upper jaw, occur as early as the deciduous dentition period. They are often not noticed by the parents until the child has a certain **"laziness to chew"**.

How jaw anomalies develop

There are congenital, inherited and acquired anomalies in the maxillofacial region.

For the most part, only the causes and effects of **"acquired"** tooth misalignments and jaw anomalies can be treated effectively using the prefabricated devices of the **OrthoPrevent Aligner®** system.

Deep bites requiring treatment develop during the transition of the teeth and should be treated in the early mixed dentition period.

The development of a deep bite is related to the position of the jaws in relation to each other: If the lower jaw is too far back during the transition of the teeth, the lower incisors extend until they touch the mucous membrane of the palate. This can lead to pressure points. Only **appliance-supported bite raising** can eliminate the discomfort.

The OrthoPreventAligner® DB-W

The **OrthoPreventAligner® DB-W** (DB = Deep bite) for lifting pronounced deep overbites and the simultaneous straightening of the incisors is made of highly elastic silicone. In order to provide reliable resistance to the bite on the incisors, the **OrthoPreventAligner® DB-W** has a slightly higher degree of hardness than the other appliances in the system. The lip shield for the upper jaw is slightly steeper in order to allow the upper incisors (which are usually inclined towards the palate) to straighten.

The effect of the **OrthoPreventAligner® DB-W** is indirect and is triggered by stimulating the vertical growth of the posterior teeth according to the bite elevation principle of functional orthodontic appliances (activator, bionator, function regulator, etc.).

In the case of the **OrthoPreventAligner® DB-W**, only the incisors of the upper and lower jaw bite on the appliance, creating a blockage in the posterior region.

Since teeth lengthen throughout their life until they meet resistance, this is a good way to ensure that the bite of the incisors is raised.

DB-W Mixed dentition
(Similar to Fig.)

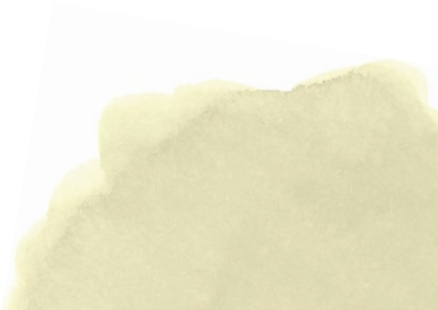
Development of the extreme deep bite

When teeth normally bite together, the incisors separate by the upper teeth biting over the lower teeth and touching each other at the back of the upper teeth - like scissors.

However, if the lower jaw is slightly back when the teeth break through, the incisors continue to grow up until they meet resistance. This is usually the gums of the palate.

Especially when the lower jaw is set back, which is present to varying degrees in about 25% of all people, a deep bite can occur involving contact with the mucous membrane.

Recommended wear

In order to break harmful habits and to eliminate **mal-functions** as well as to classify "crooked" teeth, the **OrthoPreventAligner®** should be worn regularly for at least 2 hours a day and at night.

Wearing check

	Week 1		Week 2		Week 3	
	 hr.	 hr.	 hr.	 hr.	 hr.	 hr.
MON						
TUE						
WED						
THUR						
FRI						
SAT						
SUN						

Instructions for use

Rinse the **OrthoPreventAligner**[®] with warm water before using for the first time. The **OrthoPreventAligner**[®] can be disinfected briefly by boiling water after infections.

Cleaning instructions

Clean thoroughly with a toothbrush under warm running water. Alternatively, OROFAN[®] dental cleaning tablets can be used.

The **OrthoPreventAligner**[®] is not dishwasher or microwave safe. Only clean the hygienic **OrthoPrevent Aligner**[®] box with a damp cloth.

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