- Recommended sterilization for Instruments except cutters is Autoclave. Prior to autoclaving we recommend the instruments are placed in an Ultrasonic cleaner to remove any contaminants. Place the instruments in open position in a way that they do not touch each other on a tray in order to avoid insufficient cleaning.
- Follow the Autoclave manufactures' recommendations. Autoclave for at least 30 minutes at 115°C –118°C, or 15 minutes at 121°C–124°C, or 10 minutes at 126°C–132°C. Do not heat above 132°C. Dry Heat Sterilization, Thermodesinfektor, Chemiclave and Cold Sterilization are not recommended.
- EOG (ethylene oxide gas) is also recommended sterilization.
- Sufficient drying is necessary against the rust. So, DRY Instruments completely against the risk of corrosion. The inside of Joint of Instrument needs more time to dry completely.
- Check that Instruments have been properly cleaned.

Shioda Dental Co.
53, Tobaru, Nasukarasuyama-SHI,
321-0517 JP-Tochigi

Advena Limited,
Tower Business Centre,
2 nd Flr, Tower Street, Swatar, BKR
4013 Malta

Vertrieb

World Class Orthodontics
Ortho Organizers® GmbH

Lauenbühlstraße 59
D-88161 Lindenberg/Allgäu
Telefon: +49 8381 89095-0
Telefax: +49 8381 89095-30
E-Mail: info@w-c-o.de
Web: www.w-c-o.de

12/2021

Vertrieb/Distribution

World Class Orthodontics
Ortho Organizers® GmbH

PFLEGEHINWEISE TASK ZANGEN

Instructions for
TASK instruments

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Most TASK Instruments are basically made from the martensitic stainless steel. It is the most suitable material known for the dental instruments because the proper hardness, strength and toughness can be realized by heat treatment. The stainless steel is a kind of metal alloy that forms on surface stable film effective to rustproofing but rust spreads quickly once the surface is damaged. Please care Instruments as following procedures to use for a long term.

Warning

- For intended use only by dental professionals.
- New Instruments are NOT Sterilized.
- Inspect Instruments on delivery for possible damage during transit and inspect that it functions properly.
- Before using new Instruments, it must be cleaned.

Care After Using

- Surface contamination must be removed from Instruments immediately after use.
- Check if Instruments are in good condition to use it again.
- Rinse Instruments by running water and immerse in a disinfectant solution. Washer disinfectant is not recommended to use.
- Disinfection: Glutaral is recommended to use. Do not use the following solutions. Sodium Hypochlorite, Benzalkonium Chloride, Benzathonium Chloride, Povidone- Iodine, Phenol, Formaldehyde, Chlorhexidine Gluconate, Funtinal Water (Acid Water, Oxidantal Water) & Household Detergent.
- Use a soft brush or a clean soft cloth for removal of contamination.
- Sufficient rinsing is necessary against the rust especially the joint.