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#### INSTRUMENT CARE

## **CHARACTERISTIC OF STAINLESS STEELS** —

Most TASK Instruments are basically made from the martensitic stainless steel. It is the most suitable material known for the dental instruments because the proper hardness, strength and toughness can be realized by heat treatment. The stainless steel is a kind of metal alloy that forms on surface stable film effective to rustproofing but rust spreads quickly once the surface is damaged. Please care instruments as following procedures to use for a long term.

### Warning

- For intended use only by dental professionals.
- New Instruments are NOT sterilized.
- Inspect Instruments on delivery for possible damage during transit and inspect that it functions properly.
- Before using new instruments, it must be cleaned.

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
## **CARE AFTER USE -**

- Surface contamination must be removed from instruments immediately after use.
- Check if instruments are in good condition to use it again.
- Rinse instruments by running water and immerse in a disinfectant solution. <u>Washer disinfector is not recom-</u> mended to use.
- Disinfection: Glutaral is recommended to use. Do not use the following solutions. Sodium Hypochlorite, Benzalkonium Chloride, Benzathonium Chloride, Povidone-Lodine, Phenol, Formaldehyde, Chlorhexidine Gluconate, Functinal Water (Acid Water, Oxidental Water) & Household Detergent.
- Use a soft brush or a clean soft cloth for removal of contamination.
- Sufficient rinsing is necessary against the rust especially the joint.
- Recommended sterilization for instruments is Autoclave. Prior to autoclaving we recommend the instruments are placed in an \*ultrasonic cleaner to remove any contaminants. Place instruments in open position in a way that they do not touch each other on a tray in order to avoid insufficient cleaning.
  - \*Please don't use Ultrasonic cleaner for instruments which has a blade (Pin & Ligature Cutter, Distal End Cutter, Scissors and so on).
- Follow the Autoclave manufactures' recommendations. Autoclave for at least 30 minutes at 115°C~118°C, or 15 minutes at 121°C~124°C, or 10 minutes at 126°C~132°C, or 3 minutes at 134°C. Do not heat above 135°C except items Code 10116, 10117, 10123, 10124 & 74921. Dry Heat

- Sterilization, Chemiclave and Cold Sterilization are not recommended.
- EOG (ethylene oxide gas) is also recommended sterilization.
- Sufficient drying is necessary against the rust. So, <u>DRY</u>
   instruments completely against the risk of corrosion. The
   inside of hinge of instrument needs more time to dry
   completely.
- Check that instruments have been properly cleaned.
- Lubricate periodically on all instruments with hinges, locks and the other moving parts such as forceps, pliers, scissors and needle holders. Sharpness of blades decreases by wear-out or damage. When the sharpness becomes dull, it needs sharpening or reconditioning soon.

## **Storage**

 Instruments should be stored dry, dust-free and away from chemical products.

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For improving, a design and specification are subjected to modify without notice.