OrthoPpreventAligner®

OB-**W** soft

D

GB

F

I

E

Patienteninformation

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E

Patient information

Prevention-oriented early treatment

With the **OrthoPreventAligner®** system, functional disorders such as open mouth posture, an incorrect tongue rest position and muscle-weak lip closure are corrected. If malocclusions have already developed, they are encouraged to heal themselves.

Functional disorders also include the **frontal open bite**, which results in speech disorders, restricted biting and chewing.

If permanent incisors break through irregularly during the Mixed movement of teeth, they can usually be guided into the correct tooth position with the **OrthoPreventAligner® OB-W**.

How jaw anomalies develop

There are congenital, inherited and acquired anomalies in the maxillofacial region. For the most part, only the causes and effects of “acquired” tooth misalignments and jaw anomalies can be treated effectively using the prefabricated devices of the **OrthoPrevent Aligner®** system.

These anomalies are usually caused by non-physiological influences on the developing deciduous dentition: by sucking habits and muscular dysfunctions. The causative dysfunctions and the jaw anomalies that have already developed usually transfer to the permanent dentition and can become even worse if left untreated.

The tongue plays an essential role in the development and maintenance of dental anomalies: If the tongue is displaced from its resting position on the upper jaw or becomes lodged between the rows of teeth, dental malocclusions and functional disorders develop.

For the treatment of faulty swallowing and speech errors, logopaedic therapy should also be provided.

The OrthoPreventAligner® OB-W

The **OrthoPreventAligner® OB-W** (OB = Open bite malocclusion) for the treatment of the frontal open bite is made of a highly elastic silicone. The lip shield for the upper jaw is extended and strongly tilted backwards. As a result, the shortened incisors and the curved jaw section are fully enclosed.

The effect of the **OrthoPreventAligner® OB-W** in eliminating the open bite is indirect: The tongue is prevented from permanently positioning itself between the incisors. This allows the teeth to lengthen again in a self-regulating manner. In addition, there is a direct effect by placing the upper lip shield on the usually protruding and shortened teeth, which are thereby directed towards the palate and simultaneously lengthened.

By closing the open bite, the defective phonation is corrected, pronunciation is improved and biting with the incisors is possible again.

OB-W Mixed dentition

In the case of an open bite with
prolonged vestibular pelotte
(Similar to Fig.)

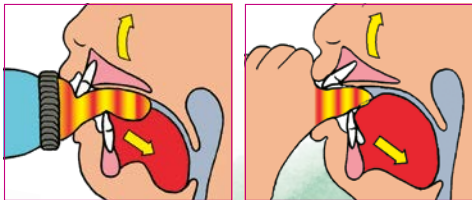
About habits and malfunctions

If the incisors do not bite over each other like scissors and there is a more or less large gap between the upper and lower teeth, this is called a **frontal open bite**. This malocclusion is already acquired in the deciduous dentition - one can say that it is self-inflicted - and is only congenital in rare cases.

The open bite is primarily caused by the persistent sucking of the thumb, soothers or other foreign bodies beyond the age of 3. The continuous pressure of foreign objects especially shortens the upper incisors and bends the front part of the upper jaw. Self-healing is usually not possible after the child has stopped sucking after the age of 4 or 5. This is because the tongue is constantly placed between the shortened teeth when swallowing and speaking. As a result, the open bite remains even after the permanent teeth have broken

through and can only be eliminated with aids such as the **OrthoPrevent Aligner® OB-W** or other more complex devices.

Recommended wear

In order to break harmful habits and to eliminate **mal-functions** as well as to classify "crooked" teeth, the **OrthoPreventAligner®** should be worn regularly for at least 2 hours a day and at night.

Wearing check

	Week 1		Week 2		Week 3	
	 hr.	 hr.	 hr.	 hr.	 hr.	 hr.
MON						
TUE						
WED						
THUR						
FRI						
SAT						
SUN						

Instructions for use

Rinse the **OrthoPreventAligner**[®] with warm water before using for the first time. The **OrthoPreventAligner**[®] can be disinfected briefly by boiling water after infections.

Cleaning instructions

Clean thoroughly with a toothbrush under warm running water. Alternatively, OROFAN[®] dental cleaning tablets can be used.

The **OrthoPreventAligner**[®] is not dishwasher or microwave safe. Only clean the hygienic **OrthoPrevent Aligner**[®] box with a damp cloth.

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